# Instructions for Use

## MED-RX® SA5200 Safety Drain Line Adapter

**Important - The Safety Drain Line Adapter must be connected to the drain line prior to accessing the catheter.**

Single Use

STERILE

EO

Ethylene Oxide Sterilized

Non-Latex

### Step 1 - Attach the Safety Drain Line Adapter to the drain line

- After properly preparing the PreVac drainage bottle or bag by completely closing the roller clamp on the drain line, remove the blue cap from the male luer connector on the drain line tubing. Do not discard.
- Hold the female luer end of the Safety Drain Line Adapter in one hand and the blue locking collar of the drain line tubing in the other.
- Attach the male luer connector on the drain line tubing to the female luer connector on the Safety Drain Line Adapter by firmly pushing the ends together and turning the blue locking collar clockwise to tighten. Do not over tighten.

### Step 2 - Attach the Safety Drain Line Adapter to the Catheter

- Hold the Valve Collar and the white Safety Stylet tip of the Safety Drain Line Adapter firmly in each hand.
- Align the white Safety Stylet Tip of the Safety Drain Line Adapter straight on with the opening of the Catheter Valve.
- Introduce the Safety Stylet Tip of the Safety Drain Line Adapter into the Valve Collar.
- Advance the Safety Stylet Tip firmly forward until the silicone guard is flush against the Valve Collar. Do not advance the Safety Stylet Tip any further.

Product now ready to use. Please continue with the standard instructions to complete the drainage for this patient.

### Step 3 - Remove the Safety Drain Line Adapter from the Catheter

- After drainage is complete and the roller clamp has been closed, firmly grasp the Valve Collar and the white Safety Stylet tip of the Safety Drain Line Adapter.
- With slight effort, detach the hubs by pulling straight back. If necessary, slightly twist the Safety Stylet Tip while pulling back to assist in detaching the Safety Drain Line Adapter from the Catheter.

### Step 4 - Remove the Safety Drain Line Adapter from the drain line

- After the Safety Drain Line Adapter has been detached from the Catheter, firmly grasp the blue locking collar from the drain line tubing in one hand and the female luer end from the Safety Drain Line Adapter in the other.
- Disconnect the drain line from the Safety Drain Line Adapter by turning the blue locking collar counter-clockwise and pulling straight back.
- Replace the blue cap on the male luer connector on the end of the drain line tubing.

### Step 5 - If a second bottle is needed to complete the drainage

- If a second bottle is needed to complete the drainage, open a second sterile Safety Drain Line Adapter and a second Sterile Bottle.
- Do not connect to a Drain Line Adapter that has already been used. Repeat Steps 1 through 4.

## Disposal

Follow hospital biomedical waste disposal policy.

**Canadian Hospital Specialties Limited.**  
2760 Brighton Road, Oakville, ON L6H 5T4 CANADA  
[www.chsltd.com](http://www.chsltd.com) | ph. 1-800-461-1423

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